

## Return Demonstration Check-off – Maxi Move

**I have successfully completed the procedures for operating the Maxi Move. I have demonstrated the tasks and understand that the Maxi Move is to be used to comply with the policy and procedures of the “Safe Patient Handling” program.**

**Employee Name (Print)** \_\_\_\_\_ **Date** \_\_\_\_\_

**Signature** \_\_\_\_\_ **Unit Name** \_\_\_\_\_

**Coach** \_\_\_\_\_ **(Performing Check-off)**

**ALL the questions below are safety-related and must be answered correctly!**

- The maximum weight capacity of the Maxi Move is - 440 lbs. or 503 lbs.? \_\_\_\_\_
- The Maxi Move is designed for minimal, moderate or maximum assist? \_\_\_\_\_
- Does the Maxi Move design always require the use of two caregivers? Yes No
- Can the Maxi Move be used with either the clips (with the standard Dynamic Positioning) OR the loop slings with the loop spreader bar? Yes No
- Should the two plastic inserts at the head of the sling always be inserted before use? Yes No
- Is the sitting position in the Maxi Move the most comfortable position for the patient? Yes No
- Is the equipment designed so that the patient should place their hands on the spreader bar when being lifted? Yes No
- The patient can be moved in the Maxi Move with the chassis legs either open or closed. True False
- It is important to remember not to lower the spreader bar directly on the patient. True False
- The powered DPS, if installed, is waterproof up to the blue and white label. True False
- Using a sling one size smaller is useful when lifting from the floor. True False
- If the scale unit displays “MAX”, it indicates that the unit is overloaded and the patient should immediately be lowered onto a bed or a chair. True False

<b>Maxi Move Lift</b>	<b>Completed</b>
1. Demonstrate how to lock and unlock the wheels and replace the batteries.	
2. Point out the following parts of the Maxi Move: Emergency Lower Switch, Battery Light Indicator, Mast Control Buttons and Hand Control.	
3. Show how to remove and replace the plastic “stays” of the sling (for head support).	
4. Apply a sling to the Transfer Mobility Coach® in a sitting position, verify the clips (or loops if using a loop spreader bar) are positioned correctly at the shoulders and legs, position hanger bar, hook sling to the Maxi Move, transfer the Transfer Mobility Coach® to a bed and remove the sling.	
5. Apply a sling to the Transfer Mobility Coach® in a lying position, verify clips (or loops if using a loop spreader bar) are positioned correctly, hook it to the Maxi Move, transfer to a chair and remove the sling.	
6. Lift the Transfer Mobility Coach® off the floor using the Maxi Move.	