

Smooth Moves Infection Control & Laundry Tips

1. Reusable Maxislide sheets, slings, Hovermatt mattresses and gait belts require a wipe-down between patient contacts with PDI, Sani-Cloth Plus germicidal disposable cloths.
2. Check reusable items before every use to insure the item is clean and safe for use.
3. To request distribution/replacement of reusable slings and gait belts contact MCSS, 343-7388.
4. HoverMatts can be wiped down with Sani-Cloth wipes and left to air dry (like you would wipe down a stretcher). If they are too soiled to wipe down with the Sani-Cloth, please place them in a yellow linen bag with **YOUR UNIT NAME on the outside of the bag** (safety pin or tape the name on the bag) and **CALL 3-9431** and ask for a HoverMatt pickup. Please **DO NOT** put the mats in with the regular linen. They are getting lost and not returning to the units. If you follow the above process they will deliver the mats back to you within **three days**.
5. Place soiled (reusable) slings, gait belts, Hovermatt mattresses and MaxiSlide sheets into yellow linen bag, leave in soiled utility room. **No exceptions.**
6. Disposable "Flite" slings are located in the unit Service Centers as a patient chargeable item. MCSS will keep surplus and replenish as with other unit service center chargeable items.
7. Maxislide sheet par levels are placed in blue plastic container on linen carts. The par levels are: regular (purple) MaxiSlide 10 sets and bariatric (orange) MaxiSlide sheets 3 sets.
8. Linen Services (343-9431) will replace Maxislide sheets to par levels on each cart daily.